WITNESS® Parvo

CANINE PARVOVIRUS ANTIGEN TEST KIT

WITNESS® Parvo

ENGLISH

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I. MATERIALES
Canine fecal material is required for this test.

IV. SAMPLING
Stool samples should not be stored longer than 24 hours after collection, if stored at room temperature.

If testing is delayed, samples should be kept refrigerated (up to 7 days at +2 °C to 8 °C, 5° to 46 °F).

For long term storage, samples should be kept frozen (-20 °C or 4 °F or colder).

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Avoid touching or damaging the membrane in the sample wells or the results windows.

The WITNESS® device should be placed on a flat, horizontal surface while performing the test.

Use a separate extraction device for each sample.

Hold extraction device and buffer bottle vertically when dispensing sample and buffer respectively.

Do not touch the extraction device or buffer bottle.

Extraction device must be stored in a dry, airtight container.

For extraction use only.

VII. TEST PROCEDURE

Importante: Allow sample drops to fall onto the membrane at window (1). Do not touch the device tip or the sample drops directly to the membrane.

1. SAMPLING COLLECTION

- Remove the top portion containing the cotton swab, from a sample extraction device.
- Dispense Extraction buffer into the base of the sample extraction device until the extraction probe is visible.
- Take the top portion of the sample extraction device, coat the cotton swab with a thin layer of feces from a stool sample, or a rectal swab.

• Insert the swab into the dispensing orifice of the base of the sample extraction device. Securely fit the top portion onto the base and tap to mix for 5 seconds to ensure good sample extraction.

2. SAMPLE APPLICATION

- Tap the top portion of the sample extraction device in a flat horizontal surface.
- Dispense the base portion of the sample extraction device (at the blue line) containing the extracted fecal sample.

• Invert and hold the sample extraction device in a vertical position and squeeze the base gently to dispense five drops of sample to the test well.

• Invert the test device for final migration of sample/reagent complex.

3. READING TEST

After 10 minutes, observe the presence or absence of pink/red bands in the reading windows (2) and (3).

If a pink/red band is present, do not read the test as complete; it is a test for a test.

VIII. RESULTS

Test is valid if a pink/red band is present in reading window (3).

Interpretation of results:

- Positive: One band in reading window (2), with a band in window (3); sample is positive for CPV antigen.
- Negative: No band in reading window (2), with one band in window (3); sample is negative for CPV antigen.
- Invalid: No band in control window (3).

• Samples with low levels of CPV antigen may produce a lighter colored yet distinct pink/red band.

A test result should always be interpreted in the context of available clinical information and history for the dog being tested. It is recommended to retest using a fresh sample.

SYMBOLS DESCRIPTION

Use by Date

ECREP Authorized Representative in the European Community

LOT: Batch Code

SN: Serial Number

Temperature Limitation (Storage temperature range)

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Zoetis Italia Srl